




INSTRUCTIONS FOR USE


Luer lock* connector Syringe pre-filled with BIOBank bone powder

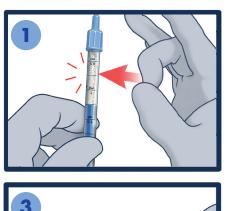

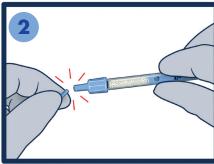



- Pull the plunger back (≈ 0.25 ml) and tap the syringe to decompress the powder
- Break off the breakable tip of the plug

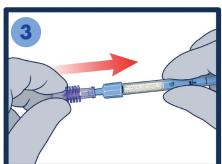

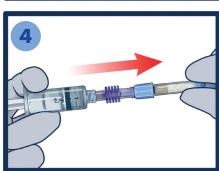



- 3 Connect the provided Luer lock connector to the syringe filled with powder
- 4 Connect a syringe filled with a sterile sodium chloride solution.
 Aseptically transfer the liquid to the plunger to completely hydrate the powder

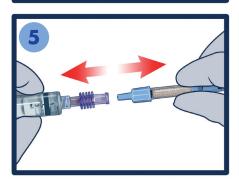

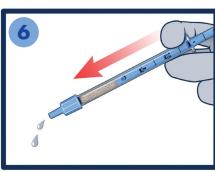

- 5 Disconnect the BIOBank syringe
- Wait few minutes then evacuate the excess liquid by pushing the plunger

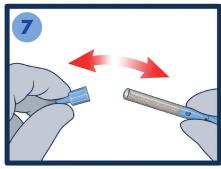

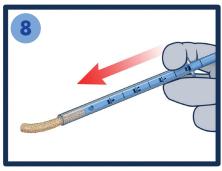

- **7** Remove the plug
- 8 Push the plunger to inject the rehydrated BIOBank bone powder into the graft site

*The Luer lock connector is a sterile medical device provided as an accessory

3 rue Georges Charpak - 77127 Lieusaint - France

Orthopaedics tel.: +33 1 64 42 59 66 - Dental tel.: +33 1 64 42 59 65 Fax: +33 1 64 42 59 60 - commercial@biobank.fr - www.biobank.fr